

# DeRoyal® PRUventor™


## APPLICATION GUIDE



The PRUventor™ has a soft interior filling that when combined with the unique fabric, creates a fluid-like sensation. Therefore, pressure around the bony prominences of the ankles and metatarsals is reduced.


### APPLICATION INSTRUCTIONS

step 1

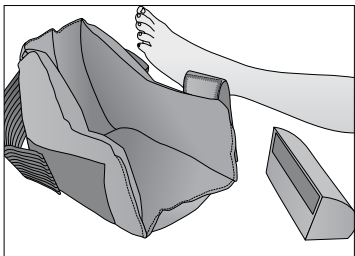


Release all hook and loop closures.

step 2

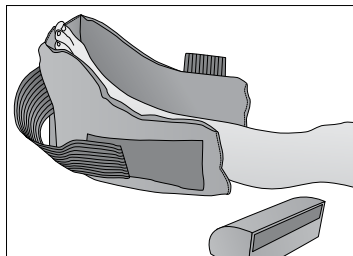


Gently place the patient's foot into the boot making sure the heel is placed over the heel opening.

step 3

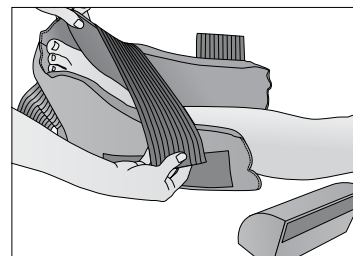


Take one strap and bring it over the top of the foot and secure the hook tab to the loop on the opposite side of the boot. Take the opposite strap and cross it over the first strap and secure the hook tab to the loop.

step 4

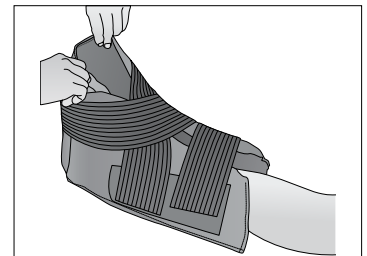


Pull the foot flaps up to ensure the foot is secure inside the boot.

step 5

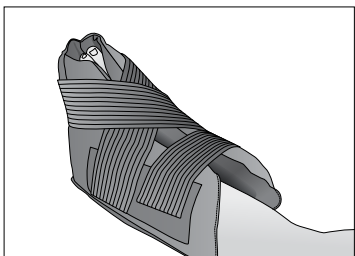


If using PRUventor™ Long, apply the leg strap and secure hook tabs to the loop on both sides of the boot. **NOTE:** Not applicable for PRUventor™ Short.

step 6

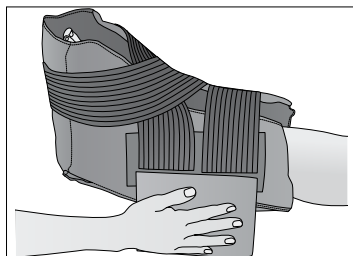


Apply the anti-rotation wedge by attaching the hook side of the wedge to the exterior, lateral side of the boot. Make sure the bottom of the wedge sits flat on the surface of the bed.

step 7

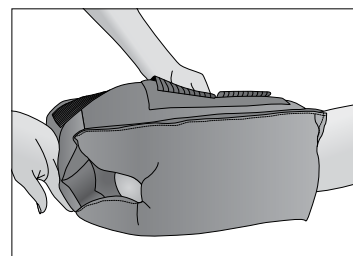

Visually check periodically to ensure the heel is suspended over the heel opening. If not, remove the boot and reapply as necessary.

step 8

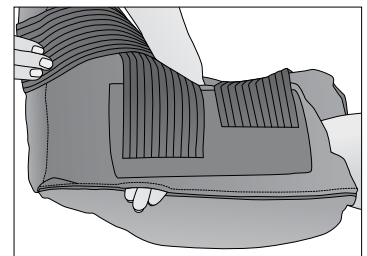

If the PRUventor™ is used with sequential or intermittent compression devices, make sure tubing is not bent and is not in contact with the patient's skin. Tubing should exit through the nearest available port. **DO NOT** use the heel opening as an exit for tubing.



Improving Care. Improving Business.®